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Instructions for use

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Material-related characteristics

Chemical composition [wt. %]

Polymethylmethacrylate (PMMA)	> 99
Coloring pigments	< 1

Physical specifications

Flexural strength*	[10477]	[MPa]	96
Flexural strength	[20795-1]	[MPa]	65
Flexural modulus	[20795-1]	[MPa]	2030
Fracture toughness	[20795-1]	[MPam ^{1/2}]	2,6
Charpy impact strength, +23°C	[ISO 179/1eU]	[kJ/m ²]	45
Density	[ISO 1183]	[g/cm ³]	1.17
Water sorption	[ISO 20795-1]	[µg/mm ³]	< 24
Solubility	[ISO 20795-1]	[µg/mm ³]	< 0,3
Residual monomer content	[ISO 20795-1]	[%]	< 0.5

* determined by 3-point flexural test following ISO 10477

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Explanation of symbols

Manufacturer	Date of manufacture	Use-by date
Batch Code	Catalog number	Medical Device
Keep dry	Keep away from sunlight	
Content (Quantity)	Consult instructions for use	

R_xonly

Caution: U.S. Federal law restricts this device to sale by or on the order of a dentist.

1. Indications

DD provi P HI are pre-colored dental milling blanks made of PMMA for the manufacture of temporary crowns and bridges** for long-term application in the oral cavity up to 12 months.

2. Contraindications

- intolerance to the contained components
- bridges with two molar pontics
- too little connector cross section
- model casting with clasps, bars, narrow sublingual bars and transversal connectors

3. Safety information

Please pay attention to the information in the material safety data sheet in its current version. Avoid the inhalation of dust particles during processing. Wear protection gloves, safety glasses and a dust mask to prevent irritations of eyes, skin and respiratory system.

4. Handling and storage

Only use clean, defect-free material from undamaged packaging and make sure that the blanks are stored at room temperature, in their original carton, dry and protected from light. Avoid contact with fluids.

5. Construction

Handling this medical device should only be allowed to trained staff.

** The indication for bridges in Canada is limited to six units with a maximum number of two pontics.

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Following construction parameters need to be considered:

	Single crown	3-unit bridge	≥ 4-unit bridge
Wall thickness [mm]	≥ 0,8	≥ 0,8	≥ 0,8
Occlusal thickness [mm]	≥ 1,5	≥ 1,5	≥ 1,5
Connector cross-section [mm ²]	-	≥ 12	≥ 22

In general all constructions should be designed in a way to enable the compensation and the transfer of the mastication forces to the abutment teeth.

6. Milling and further processing

Milling must be carried out with a milling strategy adapted to PMMA.

⚠ Only use one-edged milling tools in the milling machine or wet grinding procedures!

Further processing:

Carbide cutters with FSQ toothing and black-ring cutters for titanium are suitable for roughing. The separation of the denture bases from the milling blank and the fine finishing can be performed with simple, cross-toothed carbide millers. The connectors are cut out carefully and without high pressure. The polishing is carried out in the usual manner with polishing agents customary for denture base materials. When polishing the material, ensure that the contact pressure is kept as low as possible, otherwise the heat generated can lead to smearing / melting.

7. Cleaning

- ultrasonic bath (max. 40 °C / 104 °F) with mild cleaning agent

Do not use the following methods for cleaning:

- steam blasting
- autoclave method (e.g. sterilization / disinfection)
- organic solvents and acids (e.g. acetone, hydrochloric acid)
- highly concentrated alcohols
- chemical denture cleaners
- abrasive cleaning agents

8. Mounting

Bonding extra-orally:

Choose primer suitable for the adhesive. Only use special adhesives, adapted to the material you want to combine with the PMMA. Pay attention to the instructions of the adhesive manufacturer.

Cementation:

For temporary cementations are recommended temporary cements or glass ionomer cements.

9. Disposal

Remains of PMMA blanks can be disposed of in the regular household garbage.

We reserve the right to make changes because of the continuous development of our products. Please find the current version of the instruction manual on our website: www.dentaldirekt.de/en **This version replaces all previous versions.**